



**Not Safe!**



**Not Safe!**

New, Patented **SealSafe® Adapters** make liquid medication storage and dispensing ***safer and easier than ever***

. Oral syringes effortlessly draw the exact dose every time while the contents are more protected from the risk of spill, evaporation and contamination. The SealSafe® adapter fits snugly into most bottles while allowing the bottle's original screw-cap to fit over the adapter.

To use, simply insert the tip of an oral syringe into the adapter's seal, draw a desired dose<sup>1</sup> and remove the tip. The SealSafe® adapter instantly recloses providing a safe and clean seal.



